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Toestemming van arts of zorgverlener vereist.

## Indication and Important Safety Information

gammaCore Sapphire™ (non-invasive vagus nerve stimulator) is intended to provide non-invasive vagus nerve stimulation (nVNS) on the side of the neck. gammaCore is indicated for the treatment and / or prevention of primary headache (migraine, cluster headache, and hemicrania continua) and medication overuse headache in adults.

The long-term effects of the chronic use of the device have not been evaluated.

### gammaCore contraindications include:

- Patients with an active implantable medical device, such as a pacemaker, hearing aid implant, or any implanted electronic device
- Patients who are using another device at the same time (e.g., TENS Unit, muscle stimulator) or any portable electronic device (e.g., mobile phone)

### Warnings

- Safety and efficacy of gammaCore have not been evaluated in the following patients:
- Patients diagnosed with narrowing of the arteries (carotid atherosclerosis)
- Patients with a metallic device such as a stent, bone plate or bone screw implanted at or near their neck
- Patients who have had surgery to cut the vagus nerve in the neck (cervical vagotomy)
- Pediatric patients (less than 12 years of age)
- Pregnant women
- Patients with clinically significant hypertension, hypotension, bradycardia, or tachycardia

### Precautions

- You must read the gammaCore Instructions for Use before using gammaCore
- Only use gammaCore as described in these Instructions for Use or as otherwise directed by your Doctor or Healthcare Professional
- Only use an electroCore-approved electrode gel with gammaCore.

Note: This list is not all inclusive. Please refer to the gammaCore Instructions for Use for all of the important contraindications, warnings and precautions before using or prescribing this product.

Doctor or Healthcare Professional authorisation required.

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